IMPORTANT SAFETY NOTICE: READ CAREFULLY BEFORE BEGINNING ASSEMBLY!

- * Always use proper tools.
 - * Follow the assembly steps in order. Do not skip any steps.
 - * Be sure to check all packing materials carefully for small parts may have come loose inside the carton during shipment. If parts are missing, contact the retailer from which you purchased the product to obtain any missing parts. DO NOT USE SUBSTITUTE PARTS.
 - * Periodically check to ensure that all connectors (bolts, screws, etc.) are tight.
 - * Keep the instructions for future reference.
 - * Some assembly may involve attaching glass or mirrors. Be very careful handling the glass or mirrors, as severe injury may result if the glass or mirror breaks. Never handle large pieces of glass or mirrors by yourself.
 - * When assembling products that have electrical cords (lamps, lights, motors, power strips, etc.), always (1) make sure the electrical cord is unplugged before assembling, (2) be careful not to twist or pinch the electrical cord during and after assembly, and (3) DO NOT USE AN EXTENSION CORD WITH THIS PRODUCT.
- Note: During charging, the device and the charger could heat up, this is normal. If the battery gets hotter than usual, the device may stop charging.
 WARNING: Remove all metal objects like coins and keys from the charging surface and remove any metal objects attached to your mobile phone before placing the device on the charging surface. Some mobile devices or cases may attract metal objects.
 *When assembling shelves or entertainment centers, do not stand on lower shelves in order to assemble upper shelves—use a step ladder.
 *Do not stand on tables, including but not limited to, end tables, coffee tables, and dinner tables, as tables are not to support the weight of a person.
 *Where assembly instructions state that the assembly requires two or more people, do not attempt by yourself as there is a risk of injury.

! NOTE: ASHLEY FURNITURE INDUSTRIES, LLC DISCLAIMS ANY LIABILITY FOR DAMAGES OR INJURIES WHICH MAY OCCUR DUE TO FAILURE TO PROPERLY FOLLOW ASSEMBLY INSTRUCTIONS, PROPERLY ASSEMBLE THE PRODUCT, OR PROPERLY USE THE PRODUCT.

NOTE: YOUR UNIT MAY LOOK DIFFERENT IN APPEARANCE FROM THE ONE SHOWN IN THE ILLUSTRATIONS ON THIS INSTRUCTION SHEET.

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IMPORTANT SAFETY INSTRUCTIONS

WARNING: In order to avoid property damage, serious bodily injury or death, it is important that you read, understand and follow ALL safety instructions. Please keep the safety instructions for future reference.

-  **WARNING:** Always check under and around the chair before lowering from a raised position to make sure no children, household pets or objects are under the chair. Never allow children to operate or play on, under or around the chair. Failure to clear the area under and around the chair before lowering may result in serious bodily injury, death or property damage. Always store the controller out of reach of children and unplug the chair to prevent children from using the controller.
-  **WARNING:** To reduce the risk of electric shock, always unplug the chair from the electrical outlet before moving, cleaning or servicing.
-  **WARNING:** To reduce the risk of electric shock, fire, serious bodily injury, death or property damage, only use the chair indoors in locations free of water. The chair should never be used outdoors.
-  **WARNING:** To reduce the risk of electric shock, fire, serious bodily injury, death or property damage, never plug in, use or operate the chair if the cord has been damaged or if the chair becomes wet or has been damaged. Do not use an extension cord with the chair.
-  **WARNING:** Do not use the chair to store or support any items including, without limitation, televisions or video equipment as such items can fall off the chair and cause serious bodily injury or death.
-  **WARNING:** Use the chair only for its intended use and in accordance with these Safety Instructions and all other product instructions. Failure to do so may result in serious injury, death or property damage.
-  **WARNING:** The maximum weight limit of the chair is 250 pounds. Only allow one person to sit on or use the chair and always sit on the bottom seat cushion only. Never sit on the back of the chair, chair arms or leg rest (if equipped). Using or operating chair with more than the maximum weight limit or sitting on a part of the chair other than the bottom seat cushion could damage the chair and/or cause the chair to tip over or collapse resulting in serious bodily injury or death.
-  **WARNING:** Your chair is equipped with air openings. Always keep air openings free of lint, hair or other blockage. Failure to do so may cause the chair motor to overheat resulting in fire, property damage, bodily injury or death.
-  **WARNING:** Do not operate chair near or around flammable gases (Including oxygen tanks and masks), liquids or vapors. Failure to follow this instruction may cause a fire or explosion resulting in personal injury, death or property damage.
-  **NOTE: UNIT SHOULD BE ASSEMBLED BY TWO OR MORE PEOPLE.**
DO NOT TIGHTEN ANY BOLTS UNTIL ALL BOLTS HAVE BEEN STARTED!

RTAParts@ashleyfurniture.com
www.ashleyfurniture.parts

1-844-966-0809

FABRIC CARE INSTRUCTIONS

The microsuede fabric covering this specific piece of furniture is soft to the touch and very durable for every day living. It is recommended by the fabric manufacturer to only clean this fabric with water-based cleaning agents or soap. Lightly scrubbing will help with any tougher spots that may have developed. Always check a spot of fabric in an area that is unseen to be sure cleaning agent being used will not discolor or fade the fabric's finishing. Let the air dry and vacuum for the best result.

CHAIR ASSEMBLY - CORD AND POWER TRANSFORMER CONNECTIONS

! **WARNING:** It is important that you read and carefully follow all chair assembly instructions. Do not plug the power cord into an electrical outlet until all assembly has been properly completed. Failure to follow all chair assembly instructions may cause the chair to operate improperly possibly resulting in the risk of personal injury or death.


Once the box has been removed from around the chair, you'll notice the back is lying on the top of seat. There are few items to remove from the seat portion of the chair first. Follow the steps below:

1. Remove the zip tie holding the red bag with Power Transformer and its cord that is secured under the chair.
2. Control Wand is located in pocket on the outside of the chair as shown in **Figure C**.
3. NOTICE – INSTALL FLOOR GLIDES BEFORE USE

Your lift chair has four adjustable floor glides. They are located in the owner's package and must be installed before use. These glides can be adjusted to level your lift chair. See **Figure A**.

! **!** **WARNING:** Failing to install these floor glides may result in damage to your lift chair, flooring, and may cause the chair to become unstable and fall over causing personal injury or death

FIGURE A

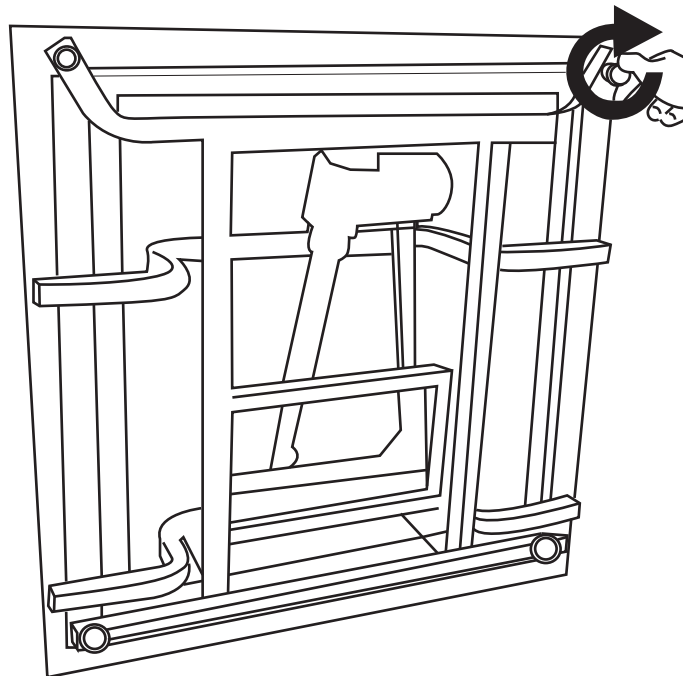4. Slide the back down on the base. The back receiver bracket slides down on the mechanism arms coming up. See **Figure B**.

5. Connect the Power Transformer cable to the corresponding plug coming from the back of the chair.

6. Power Button: The Power Button is designed to disable the power operation. It is in the cable that connects the motor to the transformer. It is a press button. When the button is pressed to the off position, the Green Light on the Power button will not display. When the button is in the on position, the Green Light in the Power Button will display and the unit will operate.

7. Connect the power cord into the Power Transformer.

8. Connect the power cord into an electrical outlet near where the chair will be used.

WARNING: Clear area below chair before lowering to reduce risk of injury or death.

If the chair is not functioning, check the plug in and connection of the cords first (The order is A-B-C), see the Figure A.

Be sure cords are clear from the chair prior to operation.

9. Press the ▲ on the wand to lift the chair and the ▼ button on the wand to lower or recline the chair.

10. Installation of back-up batteries: 9 Volt back-up batteries will operate the chair from the reclined position to the seating position one cycle in case of a power failure. If a power failure occurs and the chair is cycled, the 2 -9 volt batteries must be replaced. Unused batteries must be replaced annually.

1. Make sure unit is unplugged before proceeding.
2. Remove the sliding door on the side of the Power Transformer to locate where the two 9 volt batteries are to be installed.
3. Install two 9 volt batteries, **Batteries not included**
4. Replace door on the Power Transformer.

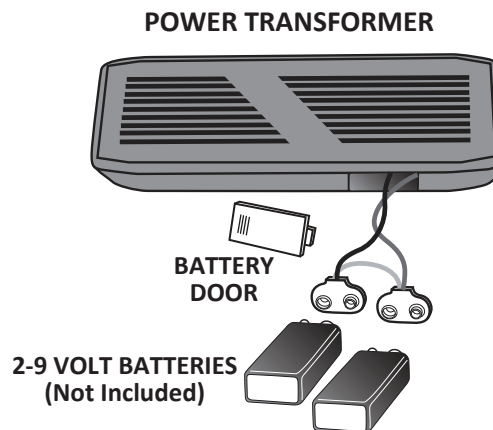
FIGURE B


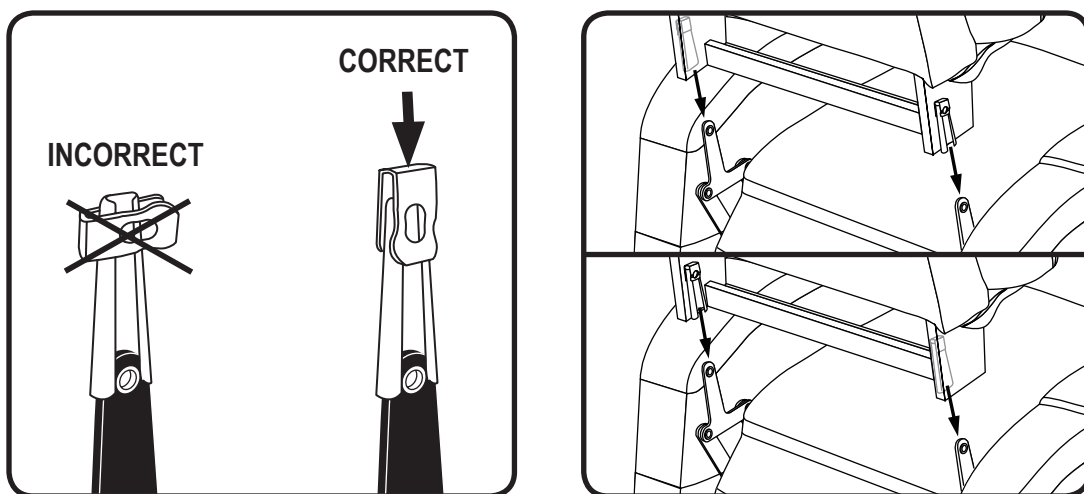
FIGURE C

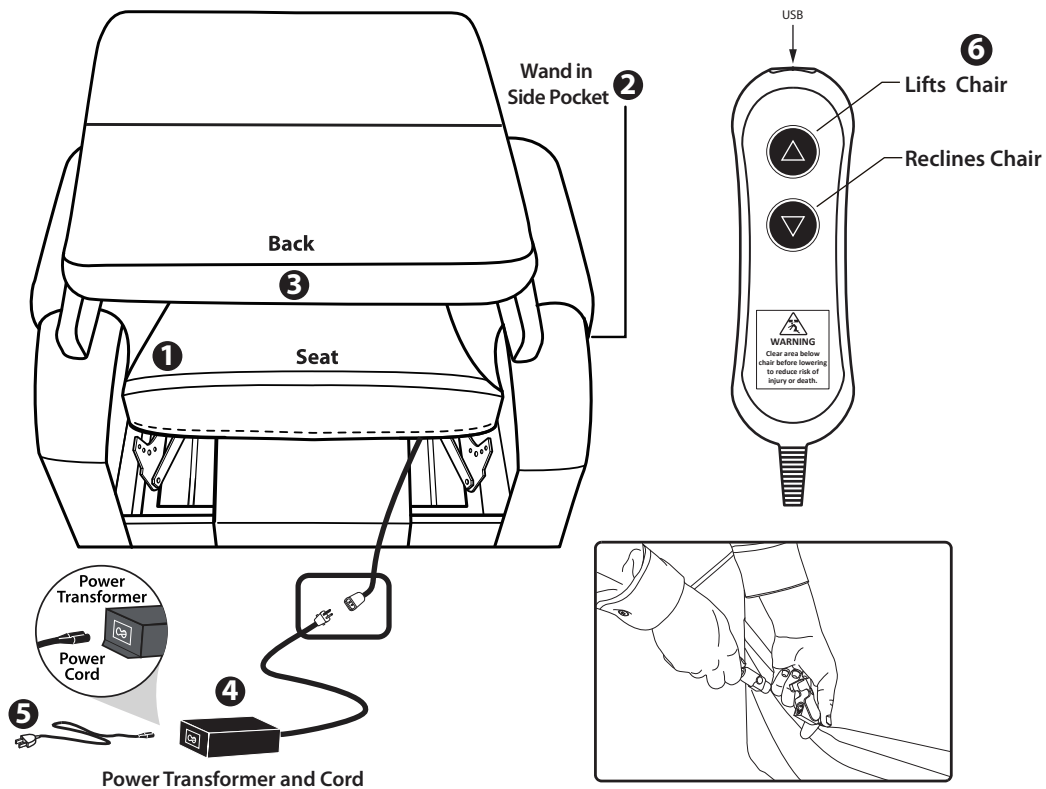HEAT AND MASSAGE FEATURE

Note: Your lift Chair may be outfitted with a heat and massage feature. If so, follow the Additional assembly and operational instructions below. Additional connections are required.

It is strongly recommended that you consult a medical professional before using the massage feature to determine if using the massage feature is right for you.

! **WARNING:** It is important that you read and carefully follow all chair assembly instructions. Do not plug the power cord into an electrical outlet until all assembly has been properly completed. Failure to follow all chair assembly instructions may cause the chair to operate improperly possibly resulting in the risk of personal injury or death.

HEAT AND MASSAGE FEATURE ASSEMBLY INSTRUCTIONS


1. Connect the Massage motor (E) from the Back to the Seat. See **Figure D**.
2. Connect the Heat pad (F) from the back to the seat. See **Figure D**.

How to use the heat and massage features

! **Warning:** It is strongly recommended that you consult a medical professional before using the heat and massage features to determine if using the features is right for you. You should turn off the heat and massage features when leaving the chair.

Press the MASSAGE button once to activate the massage function. The massage is automatically set to the WAVE setting

FIGURE D

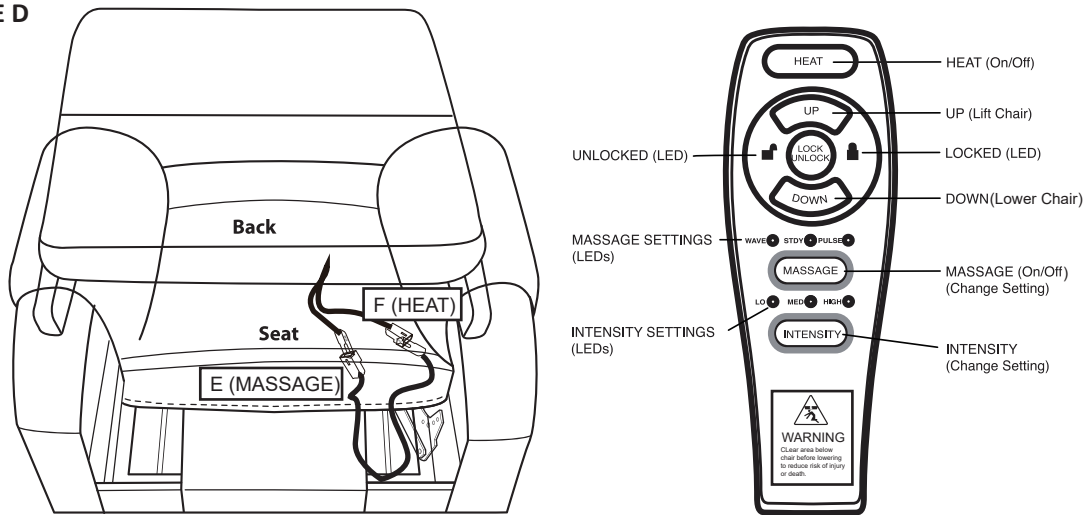Press the MASSAGE button again to go to the next of three available massage setting: WAVE, STEADY, or PULSE. Each button press will take you to next setting. Each setting is indicated by a COLORED LED light.

Press and hold the MASSAGE button for approximately 2 seconds to shut the massage function OFF. The massage function will automatically shut off after 20 minutes of continuous use.

How to adjust the intensity of the massage function

Press the INTENSITY button to set the massage intensity to one of three available settings: LOW, MEDIUM or HIGH. Each button press will take you to the next setting. Each setting is indicated by a COLORED LED light.

How to use the Heat function

To turn on or off the heat function, press the HEAT button. There is only one setting for the HEAT function, on or off.

NOTE: Heat & Massage system is designed with an auto-shutoff mechanism that turn off after 20 minutes of use. It is recommended the heat and massage units not be used for more than 20 minutes at a time. It is strongly recommended that you consult a medical professional before using the heat and massage features to determine if using the features is right for you. You should turn off the heat and massage features when leaving the chair.

How to lock and unlock the Lift/Recline function using the wand

TO LOCK - Press and hold the LOCK button for approximately 3 seconds to lock the hand control. The LOCK indicator will light up RED to indicate the hand control has been locked. The Heat & Massage functions can be turned on or off when the LOCK function is used. The Massage function can also be adjusted when the LOCK function is used.

TO UNLOCK - Press and hold the LOCK button for approximately 6 seconds to unlock the hand control. The UNLOCK indicator will light up GREEN to indicate the hand control has been unlocked

This Class B digital device complies with part 15 of FCC rules. "Operation is subject to following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference, including interference that may cause undesired operation

To comply with the FCC RF exposure requirements, no change to the antenna or device is permitted Any change to the antenna or device could result in the device exceeding the RF exposure requirement and void users right to operate device.

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